Troubleshooting Guide

| If this happens | Cause | Try this solution |
|--|--|---|
| Device doesn't turn on or nothing visible on display. | No power. Battery power is depleted. | Ensure that the device has been fully charged. Press and hold ON/+ button for 3 seconds. |
| The battery indicated on LCD display flashes continiously. | Limited battery power. | Ensure that the device has been fully charged. |
| The device does not seem to be charging. | Limited battery power. | • Ensure the USB cable is properly connected in the device and plugged into the AC adaptor and electrical socket. |
| The device turns on, but intensity cannot be increased beyond "1" for extended period. | Auto intensity reset safety feature is initiated. | Connect lead wires to device, electrodes to lead wires and place on body part. 2 small or 1 XL electrode pad per channel is required. Replace used electrode pads. The quality of the gel may be diminished. |
| The device turns on, but does not generate electric pulses. | Lead wires or electrode pads are broken or disconnected. Treatment time expired. | Replace/reconnect lead wires. Ensure lead wires are properly seated in CH1/CH2. Switch the device to the OFF position and then power ON. Increase treatment time. |