

Therapeutic Wearable System

Step by Step Quick Start Guide

When you are familiar with the operation of the iReliev® Therapeutic Wearable System, use the following steps as a quick reference to operate the device. For more information, please refer to the instruction manual.

1. Charge the remote and the pod(s) as per the instructions.



2. Attach the pod(s) to the electrode pad(s) as per the instructions. (Page 14)



3. Place the electrode pad(s) on your body as per the illustration in the manual. (Page 19)




4. Turn ON the pod(s) by pressing the power button (3 seconds).




5. Turn ON the remote by pressing the ON Button (3 seconds).




6. Press SET and the TENS or EMS icon will blink.

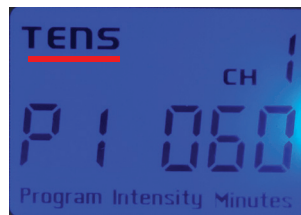


or

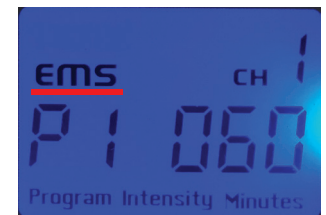

7. To select TENS or EMS, press ON/+ until desired mode is flashing, then press SET.

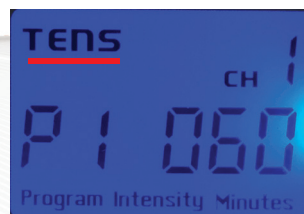


or

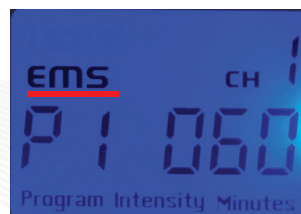


8. Now the Therapy Program icon (P1-P8 for TENS and P1-P6 for EMS) will blink. To select a Therapy Program, press ON/+ or OFF/- until desired program is flashing, then press SET.



9. Now the Treatment Minutes icon will blink. To select Treatment Minutes, press ON/+ or OFF/- until desired treatment minutes is flashing, then press SET.

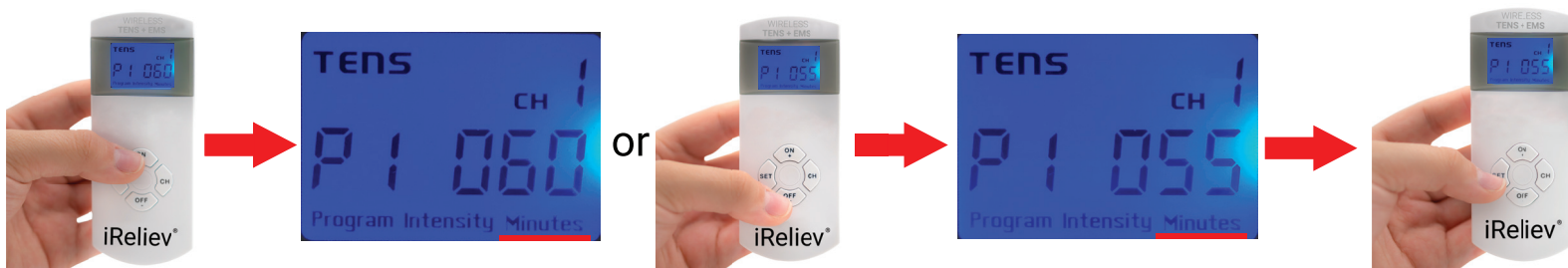

10. Now the intensity level icon will blink starting at "0." To increase the intensity level, press ON/+ gradually until you feel a comfortable tingling sensation, then press SET.

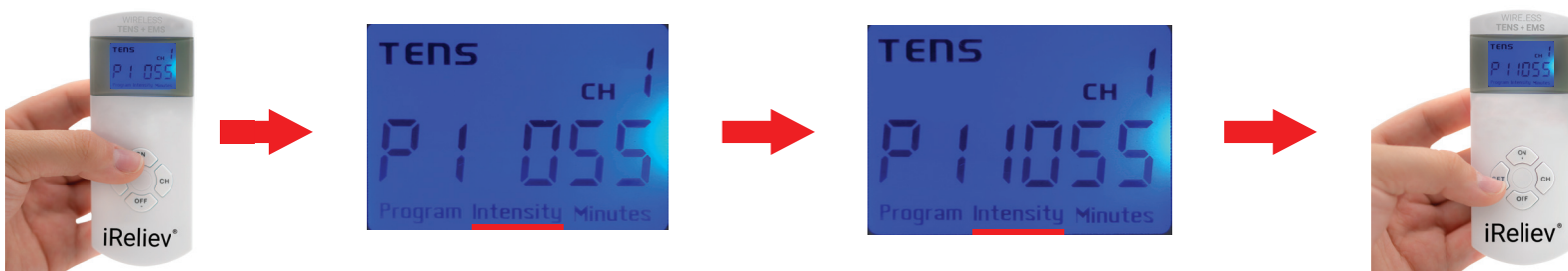

11. Stimulation will be applied until the preset time has elapsed. To turn off the receiver pod(s) when treatment minutes are remaining, reduce intensity to "0", and then press and hold the OFF/- button for 3 seconds. When treatment minutes have not been set, press and hold the OFF/- button for 3 seconds.

Note: Intensity must be >0 for each channel pod you wish to use or pod will automatically turn off after 3 minutes.

Note: Pods will only turn off automatically under the following conditions:

1. Treatment minutes have expired.
2. No activity, or if treatment intensity has not been set for 3 minutes.

Note: The remote will display the previous chosen settings for each respective pod. To save battery power, press the OFF/- button after setting device.

Note: You may go through this programming sequence with the pods attached to the electrode pads on the provided clear plastic shields. Please do so on a flat surface.